

# acton™

Topical Delivery System

## Quick Reference Guide



**Warning:** Read Instructions for Use before applying the Acton Topical Delivery System

**Rx Only**

Caution: Federal law restricts this device to sale by or on the order of a physician.



**Single use only.**

LBL 006-01

## Quick Reference Guide

1. Clean and prepare the wound and peri-wound as appropriate. Consider the use of a skin barrier for peri-wound protection.



2. Trim the dressing ensuring that it extends 1 cm beyond the edge of the wound.

3. Apply the dressing to the wound assuring that it is appropriately positioned and aligned.



4. Secure the dressing to the patient using the provided Dressing Application Tape


5. Connect the dressing to the Delivery Port of the Fluid Delivery Unit.

6. Fill the syringe with the appropriate volume of selected solution. Trim Dressing:  
 a. Less than 16 squares fill syringe with 10.4 ml.  
 b. Greater than or equal to 16 squares fill syringe with 12.0 ml.



7. Insert the Flow Activator fully into the pump to activate the unit.

8. Connect the Syringe to the Fluid Port. Position the Fill Port lower than the Delivery Port to purge air while filling.

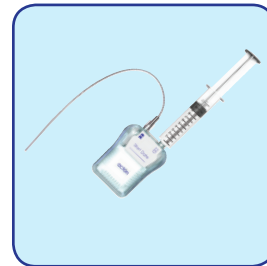

9. Expel the entire contents of the syringe slowly, filling the unit and providing a load to the dressing.

10. Monitor the wound in accordance with standard clinical practice.

Manufactured for  
**Aplion Medical Corporation**  
 1290 West 2320 South, Suite G  
 Salt Lake City, UT 84119  
 Phone: 801-975-0521  
 customercare@aplionmedical.com

**aplion**<sup>®</sup>  
 Wound Care Technology